




**User Manual**

**Oldelft Motorized Multiplane TE Probes**

Type number 171M-

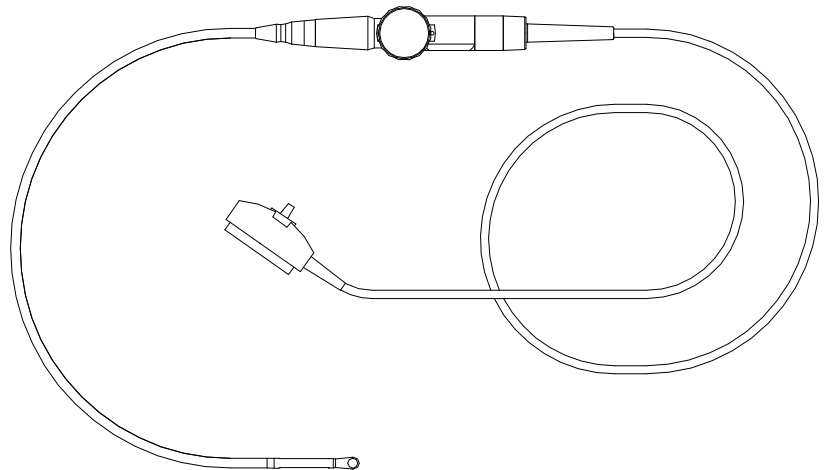

**WARNING / CAUTION**



- Before using the equipment, carefully study this manual. Keep this manual at hand for quick reference. Reread it periodically for warnings, safety precautions and maintenance requirements.
- **A damaged probe may cause an electrically hazardous condition when coupled to the human body.**
- **Do not use a damaged probe.**
- The multiplane TE probe is a precision instrument, which must be handled with care. It may be damaged when dropped or abused. In particular, do not allow the ultrasonic window in the tip to come into contact with a sharp object. Do not touch this window unnecessarily. Never exert force onto the window.
- **Only a qualified physician who has received appropriate training should perform examinations with the probe.**
- The connector is not watertight, and should always be kept dry. The control unit, although spray-watertight, should not be immersed.
- **This equipment contains no operator serviceable components. To prevent electric shock, do not remove any covers or panels.**
- **Inspect the probe for damage, cracks or bite-marks prior to each use.**
- Before introducing the probe: do not rub or spray the tip of the probe with an anaesthetic agent.
- **Please, do use the bite-guard, supplied with the probe.**
- Only use water-based coupling gel.
- As with all diagnostic ultrasound procedures keep exposure levels and duration to the minimum necessary for the examination, especially in the Doppler modes.
- **Avoid forceful manipulations and excessive force in using the probe, which could result in patient injury.**  
**Withdraw the probe only with the deflection control in the unlock mode and with the distal end of the probe straight.**
- Under normal conditions at full acoustic power the temperature of the tip does not exceed 43°C. Follow the instruction in this user manual to check this regularly.
- **Clean and disinfect as described in this user manual.**
- **Do not use methods not described in this user manual like Ethanol, Iodine, Steam, Heat or Ethylene Oxide.**
- **The up/down and/or left/right deflection may after prolonged use develop an unwanted amount of free play. In that case, contact the service organization to re-adjust the steering of the probe.**

---

## Table of Contents :


<b>1. Introduction.....</b>	<b>1</b>
1.1. Multiplane TE Probes. ....	1
1.2. Intended use of the multiplane TE Probe. ....	1
1.3. Intention of this manual. ....	2
<b>2. Preparing the multiplane TE Probe for Use.....</b>	<b>2</b>
2.1. Visual examination. ....	2
2.2. Safety precautions.....	3
2.3. Preparation for Use. ....	4
<b>3. Use of the multiplane TE Probe.....</b>	<b>5</b>
3.1. Patient selection. ....	5
3.2. Deflection and scanplane rotation control.....	5
3.3. Examination.....	8
<b>4. Cleaning and disinfecting the multiplane TE Probe. ....</b>	<b>9</b>
<b>5. Storing the multiplane TE Probe. ....</b>	<b>10</b>
<b>6. Specifications. ....</b>	<b>11</b>
<b>Annex A. Whom to Contact.....</b>	<b>12</b>




## 1. Introduction.

### 1.1. Multiplane TE Probes.

Oldelft manufactures several types of multiplane TE Probes with 32 - 64 elements, center frequency between 5 and 7MHz. All are phased array transducers, mounted in a sealed tip at the end of a gastroscope. The probes are intended for imaging of the heart through the esophagus and the stomach. The array can be rotated 180°. Scan plane rotation is controlled by two push-buttons on the control handle or via the ultrasound system. The deflection control wheels on the handle of the probe controls the deflection of the tip.

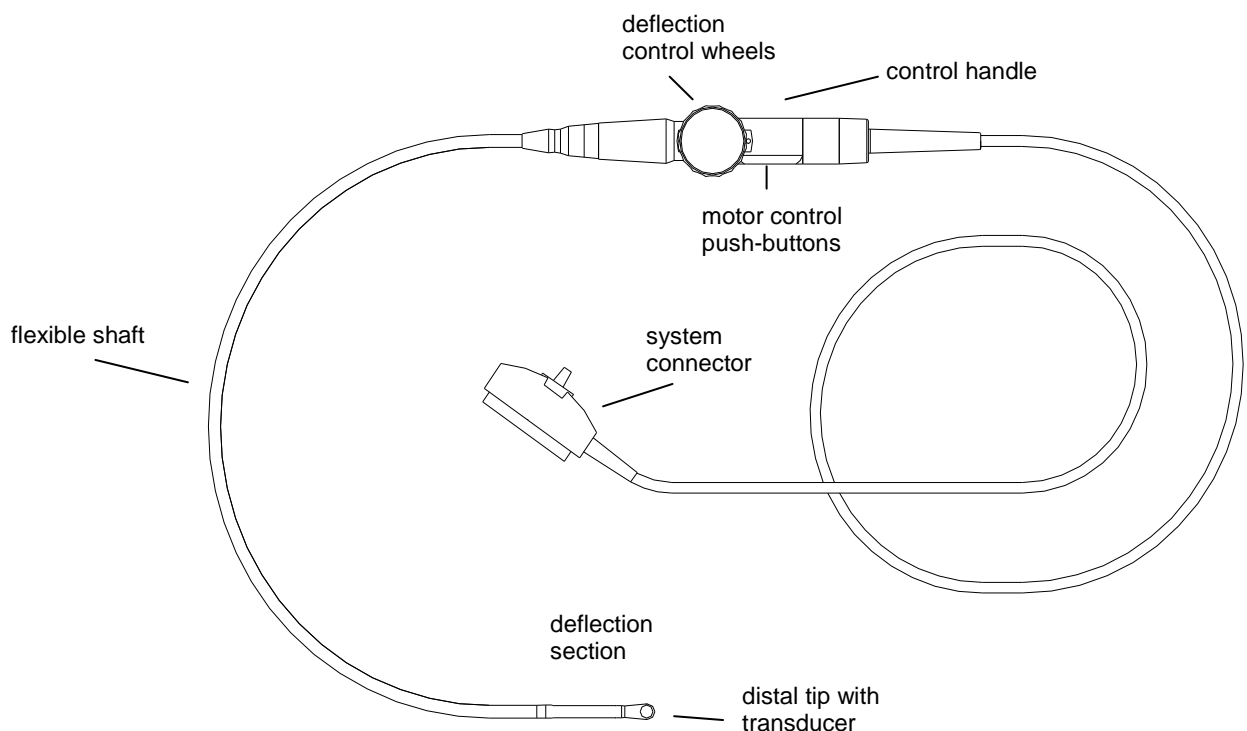

### 1.2. Intended use of the multiplane TE Probe.

**WARNING**  **The multiplane TE Probe should be used only by a qualified physician who has received appropriate training in proper operation of the probe and in endoscopic techniques as dictated by current relevant medical practices.**

The physician conducting the examination must exercise sound medical judgment in the selection of patients for this probe and be skilled in interpreting the data obtained from the examination with the multiplane TE Probe.

The probe can be used to form images of the heart to detect abnormalities, to evaluate the velocity of the blood flowing in the heart and to obtain a color depiction of the velocities of the blood flowing in the heart.

### 1.3. Intention of this manual.

This manual gives the user all information necessary to guarantee a safe and satisfactory use of the multiplane TE Probe. It also describes how to handle the probe during all stages: before, during and after the examination.

## 2. Preparing the multiplane TE Probe for Use.

### 2.1. Visual examination.

Visually examine and feel all portions of the probe before use, especially the gastroscope shaft and the flexible section at the distal end of the gastroscope. Perform the inspection of the flexible section both with the probe deflection straight and deflected. There should be no discontinuities, bumps, dents, holes, abrasions, bite-marks or any other evidence of wear or damage found.

The hard plastic section at the distal end of the probe should be smooth and firmly attached to the gastroscope shaft.

The cable and the connector that attach the probe to the ultrasound console should be free from evidence of damage.

Check the proper mechanical operation of the probe. Rotate the scanplane with the push-buttons on the handle (see §3.2). Make sure that the rotation is smooth for all scanplanes. Deflect the tip in both up/down and left/right direction using the deflection control wheel on the handle (see §3.2). Make sure the deflection operates smoothly.

**Electrical Hazard      Any evidence of damage indicates the probe can not be used and should be returned to Oldelft for evaluation and repair.**

## 2.2. Safety precautions.

- a. The multiplane TE probe is a precision instrument, which must be handled with care. It may be damaged when dropped or abused.  
In particular, do not allow the ultrasonic window in the tip to come into contact with a sharp object.  
**Do not touch** this **window** unnecessarily. Never exert force onto the window.
- b. Only a qualified physician who has received appropriate training should perform examinations with the probe.
- c. The connector is not watertight, and should always be kept **dry**.  
The control unit, although spray-watertight, should not be immersed.
- d. This equipment contains no operator serviceable components.  
To prevent electric shock, do not remove any covers or panels.
- e. Before introducing the probe: **do not rub or spray** the tip of the probe with an **anaesthetic agent**.
- f. Avoid forceful manipulations and excessive force in using the probe, which could result in patient injury.
- g. Withdraw the probe only with the deflection control in the unlock mode and with the distal end of the probe straight.
- h. Never manually deflect the distal tip of the probe ; use only the deflection control wheels.
- i. Do not kink, tightly coil, or apply excessive force on the probe cable or shaft.  
Insulation failure may result.
- j. Under normal conditions at full acoustic power the temperature of the tip does not exceed 43°C. To check that the temperature increase of the tip is within limits :
  - Connect the probe to the Ultrasound system.
  - Adjust the Acoustic Power to the highest value possible.
  - Select Color Doppler mode.
  - Wait for 2 minutes.
  - Feel at the distal end of the probe if there is a relevant temperature increase, which could be harmful for the patient.

- |  |
|--|
| <input type="radio"/> Daily              |
| <input type="radio"/> Weekly             |
| <input checked="" type="radio"/> Monthly |
| <input type="radio"/> Other              |

### 2.3. Preparation for Use.

Inspect the probe as described in §2.1.

Clean and disinfect the probe and the bite-guard as described in §4.

Make sure the system connector is plugged into the Ultrasound system and is locked by means of the locking handle on the connector.

For patient protection a sterile, single-use, latex sheath can be used.

CAUTION



Ensure that the sheath's sterile surface is maintained.

Devices containing latex may cause severe allergic reaction in latex sensitive individuals. Refer to FDA's March 29, 1991, "Medical Alert on Latex Products".

Place the bite guard on the probe so that after insertion of the probe the bite guard can easily be placed in the patient's mouth. The bite guard can also be placed in the patient's mouth before inserting the probe.

WARNING



**The use of a bite-guard is mandatory. Failure to use the bite-guard may result in damage to the probe, which could result in a safety hazard. Damage to the probe due to biting is not covered by the probe's warranty.**

Apply a sufficient amount of water-soluble acoustic coupling gel on the probe acoustic window.

CAUTION



Only use water-soluble acoustic coupling gel. Other coupling gels containing ingredients like ethanol, mineral oil, Iodine, lotions, lanolin, aloe vera or methyl or ethyl parabenoic acid can cause probe damage.

If used, place the latex sheath over the probe and gastroscope shaft up to but not covering the handle. Rub the tip carefully to ensure that all air bubbles have been removed from the probe's acoustic window area. In addition to the gel on the acoustic window, apply a sufficient amount of acoustic coupling gel on the outside of the sheath at the tip of the probe.

CAUTION



Do not exert force on the window area.

### 3. Use of the multiplane TE Probe.

#### 3.1. Patient selection.

Although the multiplane TE probe can provide clinical data not available from other instruments, there are some considerations to be made in patient selection for safe use of the probe.

**CAUTION**



The ability of a patient to swallow or accommodate the probe should be considered.

Any history of gastro-esophageal diseases must be determined and considered as well as the possible effects of other therapies the patient is undergoing.

All gastro-esophageal abnormalities must be considered as well.


#### 3.2. Deflection and scanplane rotation control.

The control handle is designed for one-hand operation. Normally the operator takes the control handle in his left hand. Thumb, first and second fingers control the deflection control wheels and scan plane rotation push-buttons.

The wheel closest to the control handle is for controlling the probe up/down tip deflection. The wheel further away from the control handle is for controlling the probe left/right tip deflection. Both wheels have a locked and freely moving mode. In the locked mode the movement of the deflection wheel is restrained. This is used to hold the tip in a certain position. A metallic ring around the body of the handle, which clicks on/off, controls the lock of the up/down tip deflection. A shifting bar in the upper knob controls the lock of the left/right tip deflection.

##### Tip deflection control.

The deflection control wheels found on the handle of the probe control deflection of the multiplane TE Probe tip.

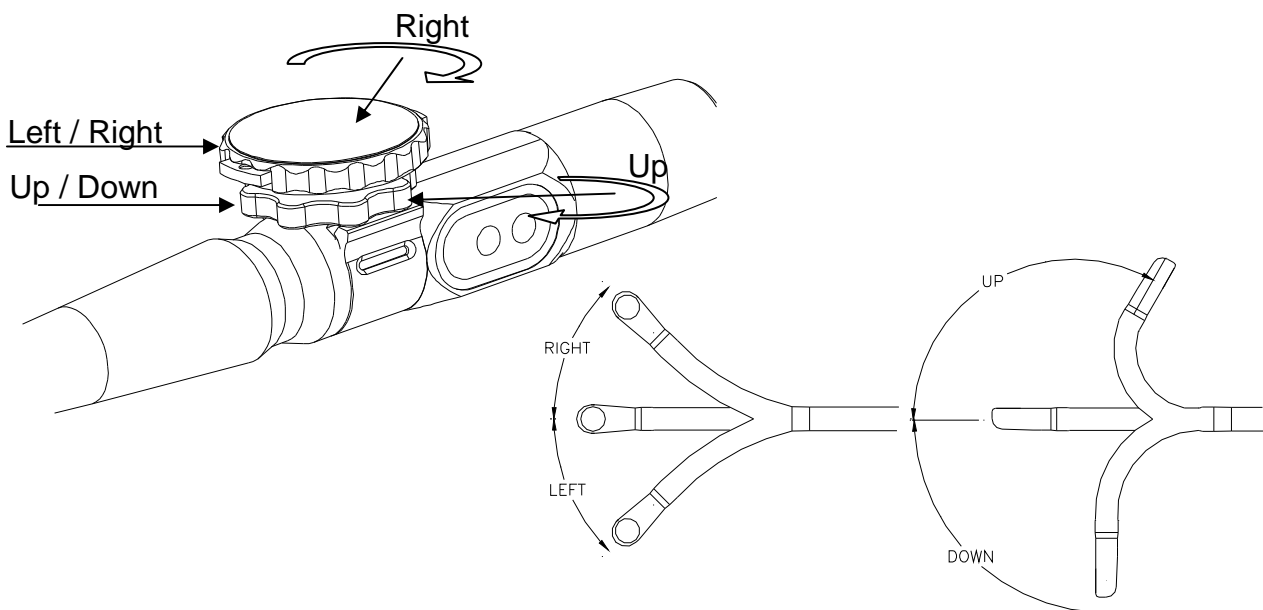

**Tip deflection control.**


**WARNING**



Check if the maximum deflection of the tip is 90° to 120° upward, 60° to 90° downwards and 30° to 45° left/right. If the deflection shows an unwanted amount of free play or exceeds the maximal deflection angles given above, do not use the probe. Contact the service organization to re-adjust the steering of the probe.


**Lock operation.**

To protect the patient and the probe, unlock the deflection controls when inserting or withdrawing the probe.

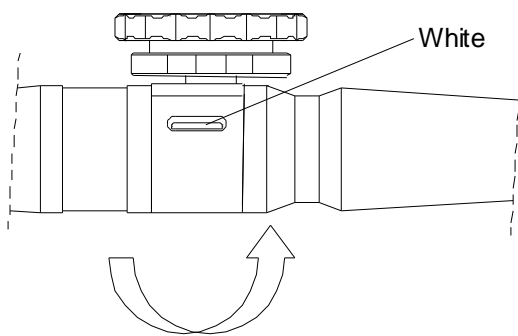

**UP/DOWN**


Unlock

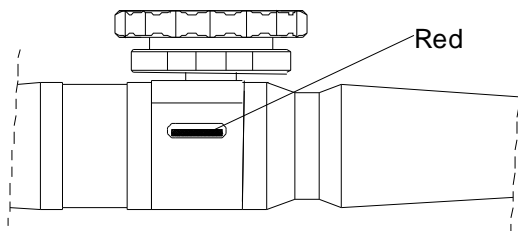


Lock

**LEFT/RIGHT**


Unlock - White

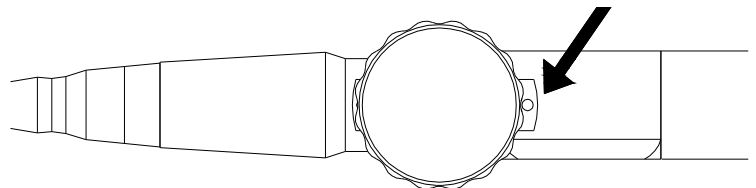


Lock - Red

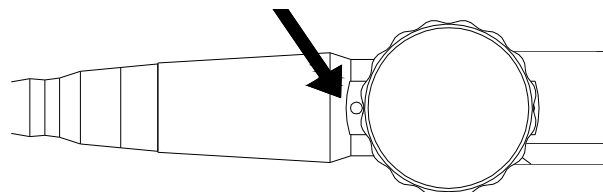

**Transducer scan plane rotation control.**

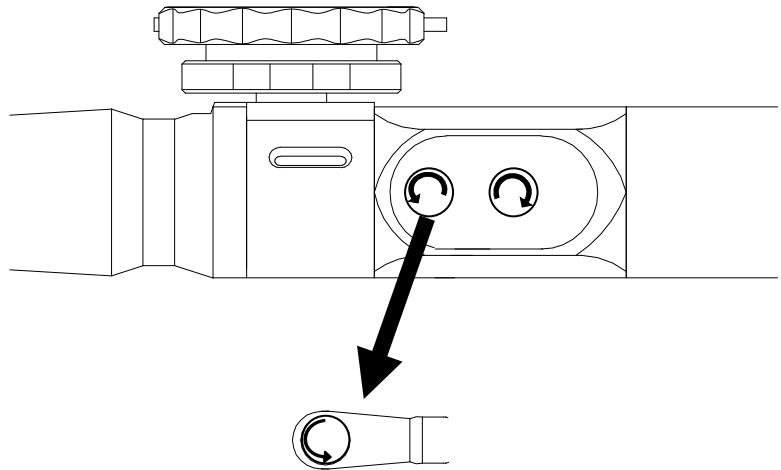

The transducer scan plane can be rotated from 0° (transversal plane) via 90° (longitudinal plane) to 180° (transversal plane, left/right inverted). All planes in between can also be chosen. This makes it possible to scan a conical imaging volume.

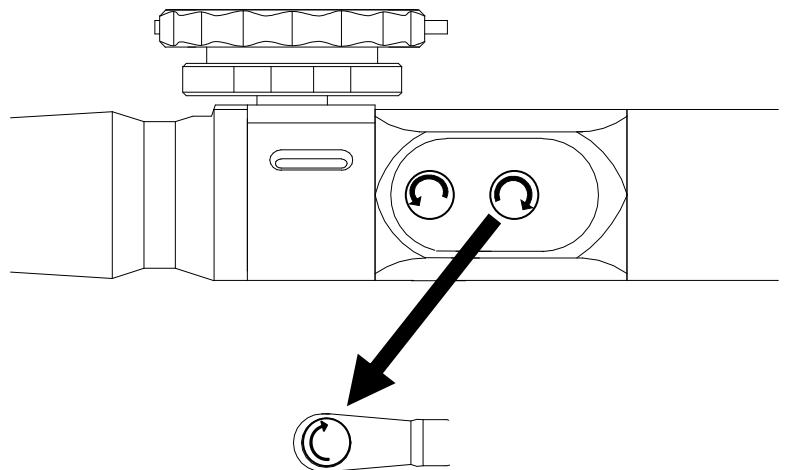

### 3.3. Examination.

The actual examination with the multiplane TE Probe is beyond the scope of this manual. There are many medical articles and books, which very thoroughly address this topic. There are however specific cautions that should be considered.

#### CAUTION



Long-term exposure to ultrasound should be minimized. Although there have been no confirmed adverse effects produced by diagnostic levels of ultrasound, unnecessary patient exposure to ultrasound energy should be avoided, especially in the Doppler mode.

Prolonged pressure on the esophagus by the tip of the probe may lead to a Pressure Necrosis phenomenon. Thus in monitoring applications the tip should be removed from the esophagus wall when you are not scanning by releasing it in the neutral position. If continuous monitoring is required the probe tip should often be re-positioned.

Whenever active scanning is not desired 'freeze' the image and release the deflection control.

Position the array such, that good acoustical contact is made. Before rotating the array, actuate the friction ('brake') on the up/down movement. If this is not done, rotating the array can mean loss of acoustical contact!

#### 4. Cleaning and disinfecting the multiplane TE Probe.

**Biological Hazard**

Adequate cleaning and, if necessary disinfection are carried out to prevent disease transmission. It is the responsibility of the user to verify and maintain the effectiveness of the procedure used. A single-use, sterile disposable sheath for TE purposes can be used.

CAUTION



Keep the control handle and system connector out of any cleaning or disinfection solutions. The control handle and cable may be cleaned with a damp cloth, but only the distal end of the probe up to the 100cm marker on the shaft may be placed into a disinfection solution.

- |                |
|----------------|
| ○ Daily        |
| ○ Weekly       |
| ○ Monthly      |
| ● Other        |
| After each use |

**Probe cleaning**

- Clean the tip and flexible shaft with gauze pads soaked in mild soapy lukewarm water (below 26°C is recommended). For other authorized cleaning agents, see Oldelft's website ([www.oldelft.com](http://www.oldelft.com)).
- Remove any materials that could interfere with the disinfection procedure.
- Rinse the distal tip and shaft with lukewarm water, in which one can hold one's hand without pain. Proceed until all visible soap residue has been removed.

CAUTION



Do not clean any portion of the probe with methanol or ethanol. Such substance can cause irreparable damage to the probe.

- Air-dry or dry with a soft cloth.

**Probe disinfection**

When necessary ultrasound probes can be disinfected using liquid chemical germicides. For other authorized disinfection liquids which have been tested for compatibility with the materials used to construct the probe, see Oldelft's website ([www.oldelft.com](http://www.oldelft.com)). Follow the disinfectant manufacturer's recommendations for solution preparation, immersion time and rinsing.

- Place the cleaned and dried probe surface in contact with the disinfection solution. Take care that only the distal end up to the 100cm marker is immersed.
- Rinse the probe tip and shaft with water and dry prior to storage.

CAUTION



The probe should never be left in the disinfection solution for more than 1 hour.

Do not forget to rinse the probe directly after disinfection.

WARNING



**Do not use other disinfection methods like Iodine, Steam, Heat or Ethylene Oxide.**

## 5. Storing the multiplane TE Probe.

The multiplane TE Probe is a sensitive and expensive instrument. Use great care when storing the probe. Make sure the probe is adequately cleaned and disinfected prior to storage.

The probe can be stored in its original case.

**WARNING**  **Avoid damage to the probe by allowing nothing to protrude beyond the case when closing the lid.**

A wall mounted probe hanger can also be used for storage. Make sure the probe control handle, cable and connector are adequately supported. The shaft should hang down freely.

The storage environmental conditions must fall within the following range :

- Temperature between  $-10^{\circ}\text{C}$  and  $45^{\circ}\text{C}$ .
- Humidity between 10% and 90%, non-condensing.

## 6. Specifications.

The probe is in conformity with the relevant provisions of the European Medical Device Directive 93/42/EEC as amended by directive 2007/47/EC. The probe is categorized in Class IIa.

The probe is classified according to IEC 60601-1 as class I, type BF, defibrillator-proof.

### **Mechanical dimensions** **171M-**

➤ Shaft external diameter	11 mm
➤ Shaft length	100 cm
➤ Probe tip width	14 mm
➤ Probe tip height	10.3 mm
➤ Length of the inflexible distal part of the probe tip	41 mm

### **Transducer**

➤ Center frequency	5.0 MHz
➤ Type	phased array
➤ Number of elements	64
➤ Aperture	9 mm
➤ Focus	50 mm

**Scanplane rotation** minimum 180°

### **Tip deflection**

Up	120° ± 10°
Down	90° ± 10°
Left	45° ± 10°
Right	45° ± 10°

### **Leakage current / Dielectric strength**

Meet the requirements of IEC 60601.

### **Biocompatibility**

All external materials of the multiplane TE Probe have passed relevant biocompatibility tests according ISO 10993.

### **Environmental requirements**

➤ Operating temperature	0 to 45°C.
➤ Storage temperature	-10 to 45°C.
➤ Relative Humidity	10 to 90%, non-condensing.

## **Annex A. Whom to Contact.**

### **Manufacturer**

Oldelft b.v.,  
Elektronicaweg 10,  
2628 XG Delft,  
The Netherlands.

Tel.: +31 15 2698 916  
Fax: +31 15 2698 905  
E-mail: [sales.ultrasound@oldelft.nl](mailto:sales.ultrasound@oldelft.nl)  
Website: [www.oldelft.com](http://www.oldelft.com)